

Lex

1 Year Limited Warranty

Your Adaptation *Lex* was carefully tested and inspected before it was shipped from the factory. We warrant this product to be free from defects in materials and workmanship under normal use and service for one (1) year from the date of purchase. In the event of a defect in materials or workmanship, we will either repair or replace without charge, at our option, any part which in our judgment shows evidence of such defect within one (1) year of purchase.

This warranty does not apply if the *Lex* has been misused, abused, altered, or tampered with. At the end of the warranty period, Adaptation shall be under no further obligation expressed or implied.

This warranty gives you specific rights, and you may also have other rights which vary from state to state.

For repair service, contact Adaptation's technical dept. to receive a required RMA# (return material authorization number) prior to the return of the device.

Adaptation, Inc.
2225 W. 50th St. Suite 100
Sioux Falls, SD 57105

(800) 723-2783, (605) 335-4445, Fax: (605) 335-4446, e-mail:
info@adaptation.com, Web Page: www.adaptation.com

Please insure your *Lex* is clean before returning it to Adaptation for service.

This is not a medical device. It should not be used where incorrect or faulty operation could reasonably be expected to result in injury or worse!

LEX

Instruction Manual



The World at a Touch

ADAPTIVATION
I N C O R P O R A T E D

2225 W. 50th St., Suite 100
Sioux Falls, SD 57105
1.800.723.2783
PRINTED IN U.S.A.

Table of Contents

INTRODUCTION	3
DEVICE OPERATION	3
LEX DIAGRAM	4
RECORDING	5
MOUNTING OPTIONS	6
TROUBLESHOOTING	7
WARRANTY	8

Trouble-Shooting

The *Lex* does not play back.

- Batteries may be too low.
- Make sure the *Lex* is in the Play mode.

The *Lex* is not recording.

- Make sure the *Lex* is in the REC mode.
- Batteries may be too low.
- Make sure to press down on the *Lex* the whole time you are recording, then release when you are done.

Messages have background noise.

- Speak clearly into the side marked MIC, holding the *Lex* about 12 inches away.
- Batteries may be too low.
- Are you near a source of electrical noise such as air conditioners, electric motors, power lines, etc.?

TalkingMyWay Communication Cards

The TalkingMyWay Object Communication Cards are a communication system for individuals whose understanding is enhanced through objects. The cards consist of an object mounted on a 4" x 6" flexible plastic card. Pairing the *Lex* with an object card is a great way to add voice to the cards. By simply attaching the object card with Velcro to the *Lex*, you have created a talking card. The *Lex* could be activated by either pressing on the card, or pulling the card off from the *Lex* will also activate the message. For more information on the TalkingMyWay Communication Cards, please refer to our catalog or to our website.

Mounting Options

When mounting on a table or other flat surface, Velcro or Dual-Lock can be used as a mounting method.

Adaptivation also has many mounting systems available. The Magic Arm, Maxess Mounting System, and solutions from Sensitrac are all great options for mounting your *Lex*. Please refer to the Adaptivation catalog or our website for more information.

INTRODUCTION


Your Adaptivation, Inc. *Lex* has been designed to provide easy access for individuals with physical or mental disabilities to basic and effective augmentative communication. The *Lex* allows an individual to record three different messages on three separate levels and play them back. Messages are accessed by pressing anywhere on the blue membrane switch on the top. Because the *Lex* is so easily programmed, it is great for promoting interaction in inclusive settings. Feel free to contact Adaptivation at any time for assistance using your *Lex*.

Operating your Lex

Battery Installation

1. Check to ensure your *Lex* is turned off by moving the MODE slide switch to the OFF position.
2. With the device lying face down, open the battery compartment.
3. Properly insert four AAA batteries. (Inside the battery compartment are "+" and "-" symbols that indicate proper polarities). Adaptivation recommends using only Alkaline batteries to ensure long battery life.
4. Close the battery compartment

Lex Diagram

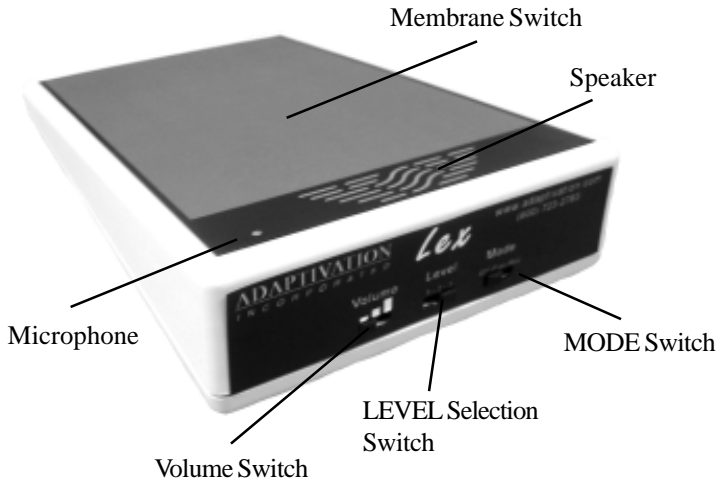

Recording a Message

1. Turn on the power by moving the Mode slide switch to the REC position.
2. Move the LEVEL slide switch to 1.
3. Press down on the *Lex* membrane and speak your message clearly, about 12 inches away, into the top of the *Lex* marked MIC, holding down until the message is finished. The message is automatically limited to a maximum of 20 seconds.
4. Upon releasing the membrane, you should hear an automatic playback of the message verifying what you just recorded.
5. To record your next level, move the LEVEL slide switch to 2, and repeat the process. Each level can be recorded independently at different times, without affecting messages on other levels.
6. When finished recording, move the MODE switch to Play. Press down on the *Lex*, and your message will play back.
7. To access the other levels, move the LEVEL switch to the desired level.

Note: When you are not using the *Lex*, we recommend turning off the *Lex* to prevent accidental activation and to save on battery life.