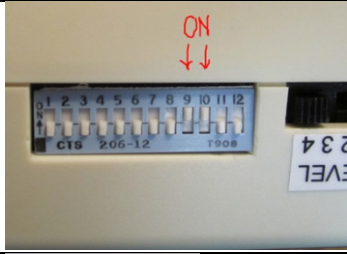

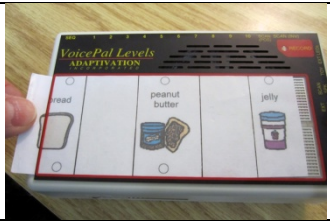

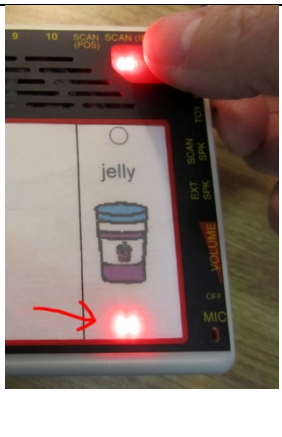





A. Recording

<p>1.</p>	<p>To begin, turn Dip Switches (DS) 9 & 10 ON, all others are set to OFF. Turn on additional Dip Switches as choices are made for the activity. <i>*DS 9 & 10 are not required for sequencing or randomizing, however, if they are on, you can still record for all types of activities and then easily switch from sequencing to auditory scanning simply by changing the placement of external switches.</i></p>	
<p>2.</p>	<p>Choose a level 1-4</p>	
<p>3.</p>	<p>Choose the number of messages (or choices) for your activity using Dip Switches 2 & 3. 2 messages = DS 2, 3 ON 3 messages = DS 2 ON (3 OFF) 5 messages = DS 3 ON (2 OFF) 10 messages = default both OFF</p>	
<p>4.</p>	<p>Slide an overlay into the overlay pouch that represents the number of choices for your activity, and plan the messages. <i>The VoicePal levels will sequence through messages, randomize them for games, or scan them for making choices.</i> <i>Tip: write the level # of the activity on the back of the overlay grid, along with Dip Switches that are ON for that activity.</i></p>	
<p>5.</p>	<p>Turn on device by turning up volume wheel.</p>	
<p>6.</p>	<p>Record your messages</p> <ul style="list-style-type: none"> Press and hold (about 3 seconds) the record area until the red light in the red record area is ON and not flashing. Press and hold the keypad area for message one and record your message. Release when finished speaking. The device will play the message you recorded when you release. Repeat for all message areas on your overlay. <ul style="list-style-type: none"> If you decide you don't like a recording, simply press and hold the corresponding message area of the keypad and re-record. When finished, press and hold the red record area until the red record light goes off. 	
<p>7.</p>	<p>To record cue words: (Tip: DS 10 must be ON) Cue words are one or two words used to represent message choices, and are heard when the device is scanning.</p> <ul style="list-style-type: none"> While in Record mode, momentarily press the red Record area again (while the red record light is on steady) and the red LED for message area 10 (bottom right of device) will light up. It can be seen through the paper overlay. This indicates that you will be recording cue words. Press and hold keypad area to record cue words for each message or choice. When finished, press and hold the red record area until the red record light goes off. 	

B. Sample Common Activity Setups

Reference:		
<p>Change # of messages Add these Dip Switch (DS) settings to setups below to change the number of messages.</p>	<p>2 messages = DS 2, 3 ON 3 messages = DS 2 ON (3 OFF) 5 messages = DS 3 ON (2 OFF) 10 messages = default both OFF</p>	
<p>Change scan speed with DS 7 & 8</p>	<p>Slowest: DS 7, 8 ON Slow: DS 7 ON (8 OFF) Fast : DS 8 ON (7 OFF) Fastest: DS 7, 8 OFF</p>	
<p>* TIP: For easy transition from one activity setup to another, keep DS 9 & 10 ON. They are not required for sequencing or randomizing, however, if they are on, you can record for and easily switch from sequencing to auditory scanning simply by changing the placement of the external switches.</p>		
Activity Setups:		
<p>Sequencing 10 messages</p>	<p>DS: All OFF* External Switch placement: - SEQ jack</p> <p><i>Sequencing occurs in any setup if dip switch 4 is OFF</i></p>	
<p>Randomizing 10 messages</p>	<p>DS: 4 ON * External Switch placement: - SEQ jack</p>	
<p>Scanning / Move and Choose 10 messages</p> <p><i>Common activity – 3 messages: add DS 2 ON</i></p>	<p>DS: 9, 10 ON External Switch placement: - Move: “SCAN (POS)” jack - Choose: “SCAN (INV)” jack</p>	
<p>Scanning / Auto scan with cue words 10 messages</p> <p><i>(3 messages: add DS 2 ON) (Slow: add DS 7 ON)</i></p>	<p>DS: 9, 10 ON External Switch placement - “SCAN (POS)” jack</p>	
<p>LED Object Scanning Place an LED on each object, up to three.</p>	<p>Plug LED lights cable into “EXT LEDs” jack on the right side of the device.</p>	<p><i>Regardless of setup, the LED’s come on only for the first 3 messages.</i></p>