

Lex
1 Year Limited Warranty

Your Adaptation *Lex* was carefully tested and inspected before it was shipped from the factory. We warrant this product to be free from defects in materials and workmanship under normal use and service for one (1) year from the date of purchase. In the event of a defect in materials or workmanship, we will either repair or replace without charge, at our option, any part which in our judgment shows evidence of such defect within one (1) year of purchase.

This warranty does not apply if the *Lex* has been misused, abused, altered, or tampered with. At the end of the warranty period, Adaptation shall be under no further obligation expressed or implied.

This warranty gives you specific rights, and you may also have other rights which vary from state to state.

For repair service, contact Adaptation's technical dept. to receive a required RMA# (return material authorization number) prior to the return of the device.

Adaptation, Inc.
2225 W. 50th St., Ste 100
Sioux Falls, SD 57105

(800) 723-2783, (605) 335-4445, Fax: (605) 335-4446, e-mail: info@adaptation.com, Web Page: www.adaptation.com

Please insure your *Lex* is clean before returning it to Adaptation for service.

8/09

Tangible Object Cards

Tangible Object Cards are a communication system for individuals whose understanding is enhanced through objects. The cards consist of an object mounted on a 4"x 6" flexible plastic card. Pairing the *Lex* with an object card is a great way to add voice to the cards. By simply attaching the object card to the *Lex*, you have created a talking card. The *Lex* could be activated by either pressing on the card, or pulling the card off from the *Lex* will also activate the message. For more information on the Tangible Object Cards, please refer to our catalog or to our website (www.adaptation.com).

Mounting Options

When mounting on a table or other flat surface, hook and loop fasteners or Dual-Lock can be used as a mounting method.

Adaptation also has many mounting systems available. The Magic Arm, Maxess Mounting System, and solutions from Sensitrac are all great options for mounting your *Lex*. Please refer to the Adaptation catalog or our website for more information.

ADAPTIVATION INCORPORATED

The World at a Touch

Lex

Instruction Manual



Adaptation, Inc.
2225 W. 50th Street, Ste 100
Sioux Falls, SD 57105
(800) 723-2783
www.adaptation.com

INTRODUCTION

Your Adaptation, Inc. *Lex* has been designed to provide easy access for individuals with physical or mental disabilities to basic and effective augmentative communication. The *Lex* allows an individual to record three different messages on three separate levels and play them back. Messages are accessed by pressing anywhere on the blue membrane switch on the top. Because the *Lex* is so easily programmed, it is great for promoting interaction in inclusive settings. Feel free to contact Adaptation at any time for assistance using your *Lex*.

Operating your Lex

Battery Installation

1. Check to ensure your *Lex* is turned off by moving the MODE slide switch to the OFF position.
2. With the device lying face down, open the battery compartment.
3. Properly insert four AAA batteries. (Inside the battery compartment are "+" and "-" symbols that indicate proper polarities). Adaptation recommends using only Alkaline batteries to ensure long battery life.
4. Close the battery compartment

Table of Contents

INTRODUCTION	3
BATTERY INSTALLATION.....	3
LEX DIAGRAM.....	4
RECORDING	5
MOUNTING OPTIONS.....	6
TROUBLESHOOTING.....	7
WARRANTY.....	8

Trouble-Shooting

The Lex does not play back.

- Batteries may be too low.
- Make sure the Lex is in the Play mode.

The Lex is not recording.

- Make sure the Lex is in the REC mode.
- Batteries may be too low.
- Make sure to press down on the Lex the whole time you are recording, then release when you are done.


Messages have background noise.

- Speak clearly into the side marked MIC, holding the Lex about 12 inches away.
- Batteries may be too low.
- Are you near a source of electrical noise such as air conditioners, electric motors, power lines, etc.?

2

7

Lex Diagram

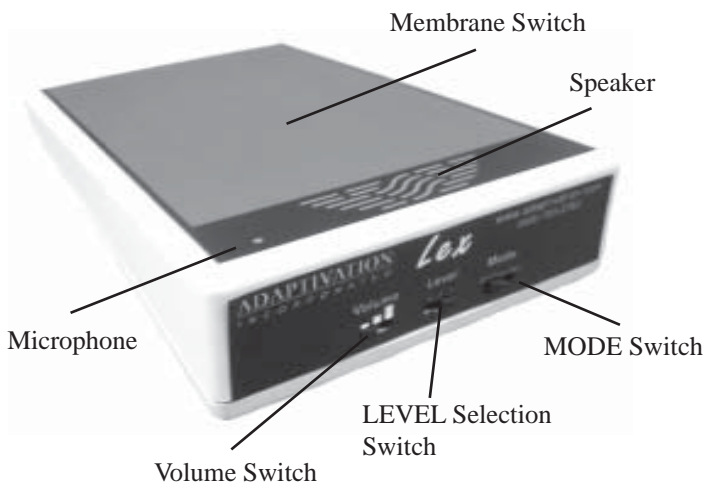

4

Recording a Message

1. Turn on the power by moving the Mode slide switch to the REC position.
2. Move the LEVEL slide switch to 1.
3. During recording, a beeping cue will be given for each message of each level. Press down on the Lex membrane and hold. When the beeping cue stops, speak your message clearly about 12 inches away, into the MIC. Continue pressing until message is finished.
4. Upon releasing the membrane, you should hear an automatic playback of the message verifying what you just recorded.
5. To record your next level, move the LEVEL slide switch to 2, and repeat the process. Each level can be recorded independently at different times, without affecting messages on other levels.
6. When finished recording, move the MODE switch to Play. Press down on the Lex, and your message will play back.
7. To access the other levels, move the LEVEL switch to the desired level.

Note: When you are not using the Lex, we recommend turning off the Lex to prevent accidental activation and to save on battery life.

5