

External Scanning Overlay



Instructions

Please Read Before Operating

By using your VoicePal Max or VoicePal Plus (scanning version only) in conjunction with your new External Scanning Overlay you can scan up to four objects or pictures.

Directions:

1. Make sure a fresh 9 volt battery (included with purchase) is installed in your External Scanning Overlay.
2. Place the black plastic overlay on your VoicePal Max or VoicePal Plus.
3. Place the lights on the object or surface that you wish to scan.
4. Set your VoicePal Max to four message mode (see VoicePal Max instructions).
5. Turn the External Scanning Overlay to the ON position.
6. Plug a Taction Pad or any switch into the scanning jack of the VoicePal Max.
7. Record the appropriate messages into the VoicePal Max.
8. Shut off the External Scanning Overlay when not being used to save on battery life.