

Taction Pad Adaptor

Congratulations on the purchase of your new *Taction Pad Adaptor*. With your *Taction Pad Adaptor* you can use a *Taction Pad* directly with any device from any manufacturer that has a standard 1/8" (3.5mm) jack, including switch-adapted toys, communication devices, computer switch interfaces, environmental controls, etc.

Also, the *Taction Pad Adaptor* can be used with Adaptiva-tion's *Flexible Switch* to improve its operation with switch-adapted toys. Some toys have high-current demands, and the *Flexible Switch* can have a hard time activating them to full speed. The *Taction Pad Adaptor* resolves this issue.

There is no ON/OFF switch for your *Taction Pad Adaptor*, so when not using it, remove the *Taction Pad* or other switch to prevent unnecessarily discharging the battery.

Replacing the Battery

You can expect many months of normal operation of your *Taction Pad Adaptor* before the battery needs to be replaced. The *Taction Pad Adaptor* uses a single CR1220 battery, commonly available at retailers such as Radio Shack, Target, Home Depot, etc, or online.

- Unplug any switch plugged into the device.
- Remove the back cover using a phillips-head screw driver.
- Lift the parts component board from the enclosure by lifting on the black cord. Handle the board **ONLY** by this cord or by the edges. ***Be careful to not disturb the protective yellow tape.***
- Remove the battery by pushing from behind with a small screw driver or other pointed tool.
- Insert a fresh battery. Observe proper polarity of the new battery. Return the parts component board to its proper location.
- Replace the back cover securely, observing proper orientation of the back cover.

Adaptivation, Inc.
2225 W. 50th St.
Sioux Falls, SD 57105

www.adaptivation.com (800) 723-2783

08/13