

ADAPTIVATION

I N C O R P O R A T E D

Personal Pager Chime/Vibrating System PP-CH/VBS

The PP-CH/VBS is the latest concept in convenience and safety for use anywhere a discrete call system is required. Now you can easily and instantly summon help or attention by simply pushing the button on the Personal Pager transmitter. The portable Personal Pager receiver will chime or vibrate, notifying the wearer that you request their attention.

- Eliminate yelling with the push of a button
- Excellent range
- Transmitter battery (12V) and Receiver batteries (2-AA) included

Instructions for Battery Installation and Use.

Pager Transmitter

1. To replace battery, remove the small screw from back of case and carefully separate the case halves. Remove and replace the small 12v battery (E23A, MS21, V23 or equivalent), carefully noting polarity. Close case halves, and insert and tighten the screw, being careful not to overtighten.
2. The red indicator light will go on when the button is depressed, indicating that the signal has been sent.
3. If you need to use an external switch to make the transmitter more accessible for the individual plug the switch into the input jack of the adapter.

Pager Receiver

1. Remove battery door and insert two AA batteries. Replace the door.
2. To activate the unit, slide the switch from the off position to either the chime or vibrate position.
3. When the button of the pager transmitter is depressed, the receiver will chime or vibrate, alerting you instantly.
4. The pager receiver can easily be positioned three ways:
 - a. Clipped to a belt
 - b. Mounted to a wall
 - c. Stood up on a table with its unique built-in easel support

NOTE:

1. Use alkaline batteries for best results
2. Switch to "OFF" when not in use
3. When the red indicator light on the transmitter fails to light, replace batteries.

Adaptivation, Inc
2225 W. 50th Street, Suite 100
Sioux Falls, SD 57105
(800) 723-2783
www.adaptivation.com