

1-Year Limited Warranty

Your Adaptivation device was carefully tested and inspected before it was shipped from the factory. We warrant this product to be free from defects in materials and workmanship under normal use and service for one (1) year from the date of purchase. In the event of a defect in materials or workmanship, we will either repair or replace without charge, at our option, any part which in our judgment shows evidence of such defect within one (1) year of purchase.

This warranty does not apply if the device has been misused, abused, altered, or tampered with. At the end of the warranty period, Adaptivation shall be under no further obligation expressed or implied.

This warranty gives you specific rights, and you may also have other rights which vary from state to state.

For repair service, contact Adaptivation's technical department to receive a required RMA# (return material authorization number) prior to the return of the device.

Adaptivation, Inc.
2305 W. 50th St. Suite B
Sioux Falls, SD 57105

(800)-723-2783, (605) 335-4445, (605) 335-4446 (fax)
info@adaptivation.com, www.adaptivation.com

Please make sure your device is clean before returning it to Adaptivation for service.

10/28/2016

ADAPTIVATION INCORPORATED

The World at a Touch

Medley

Instruction Manual



Adaptivation, Inc.
2305 W. 50th Street, Suite B
Sioux Falls, SD 57105

(800) 723-2783

www.adaptivation.com

Troubleshooting

The message is only a "clicking" noise.

- If you inadvertently bump the switch while in Record mode, the Medley will record sound for a split second which sounds like a clicking noise when you play it back.
- Check the condition of the batteries.


Your Medley seems to have quit working. It makes no sound when turned on and a switch is pressed.

- Check the batteries (they may be too weak to function or may be improperly installed).
- Check to ensure the volume is turned up.
- Completely remove batteries and wait 20 seconds before reinstalling them, then rerecord a message.
- Try plugging the switch into a different jack.
- Try a different switch.

When messages play back, they seem to be very distorted.

- Your batteries may be low. Replace if necessary.
- Adjust the volume to a lower level.
- Rerecord the messages in a quieter environment (ambient noise may be affecting the recording).
- Make sure that you are not cover the MIC while recording.

Medley Diagram

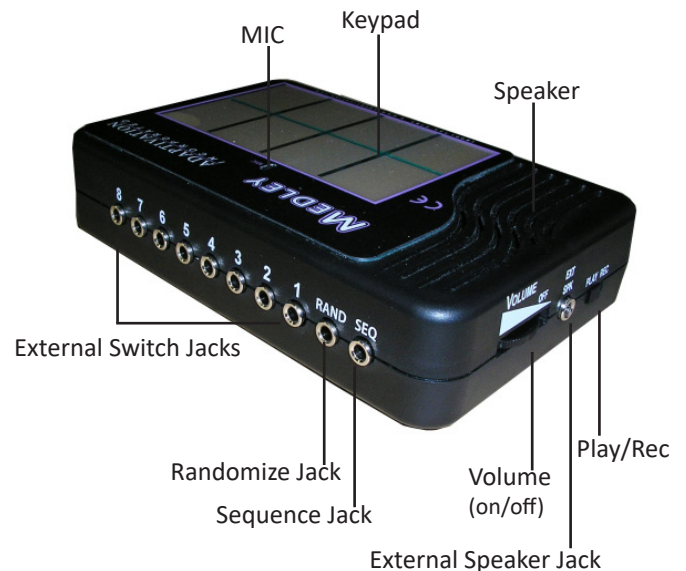

Note: The battery cover is located on the bottom of the Medley.

Go to www.adaptivation.com to download application ideas and photos of your new Adaptivation device.

Introduction

Your Adaptation, Inc. Medley has been designed to provide easy access for individuals with physical impairments and/or cognitive disabilities to basic and effective augmentative communication.

Your Medley has the capacity to digitally record up to eight 11-second messages. Messages can be accessed through

sequential messaging, random messaging, direct selection via the keypad or direct selection via external switches.

An external speaker jack is provided on the side edge of the Medley for connection to an external speaker. This is useful for connecting to an amplified speaker or P.A. system.

Battery Installation

- Check to ensure that your Medley is turned off by rotating the Volume (on/off) knob until the click is felt.
- With the device lying face down, open the rear battery compartment cover by removing the single Phillips head screw.
- Properly insert four AAA-size batteries. Inside the battery compartment the + and - symbols indicate the proper polarities.
- Put the battery compartment cover back on and secure it with the single Phillips head screw (taking care not to overtighten).
- Adaptation recommends using only Alkaline AAA batteries to ensure long battery life and consistent device operation.

2

Recording a Message


1. Turn on the power by rotating the Volume (on/off) knob to mid-position.
2. Slide the REC/PLAY switch on the end of the Medley to the REC position.
3. Press down on the keypad and speak your message clearly, about 12 inches away, into the front of the Medley, holding the keypad until the message is finished. Each message is limited to a maximum of eleven seconds.
4. Upon releasing the keypad, you will hear an automatic playback of the message, verifying what you just recorded. If you do not hear the message, rerecord the message again, insuring you are in REC mode, the batteries are fresh and properly installed, and the volume is turned up.
5. When finished recording, slide the REC/PLAY switch to the PLAY position. Press and release the switch or keypad, and your message will play back.
6. To record additional messages and play them back, repeat steps 1-5.

Note: Due to its low power consumption, the Medley can be left on for extended periods without worry about rapidly discharging the batteries, but it is best to turn the device OFF when not in use to prevent inadvertent activation.

4

Overlay Template

Feel free to make photocopies of the Overlay Template. This template can be found on Boardmaker (Mayer-Johnson), or My Pogo Boards (Talk to Me Technologies).

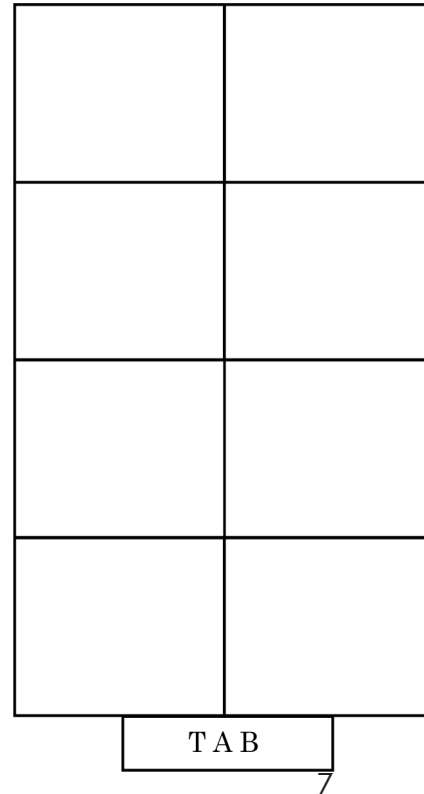

Playing a Message

Once messages have been recorded, messages can be played back by pressing the keypad or plugging in external switches.

To Direct Select: Press the keypad to play a message or plug external switches into the external switch jacks. Up to eight switches can be used for direct selection. Press the switch that corresponds to the desired message.

To Sequence: Plug an external switch into the SEQ jack. A message plays in sequential order each time the switch is pressed. To reset the sequencing order, press any keypad switch. The next time the SEQ switch is pressed, that message will be activated. Subsequent presses of the SEQ switch will play the next message in the sequence.

To Randomizer: Plug an external switch into the RAND jack. A message plays in random order each time the switch is pressed.

5